


## CROS/BiCROS Conventional Products


### UE SERIES

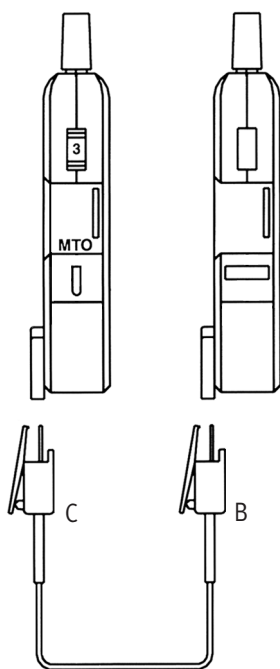

**CROS mode:**  
Slide plug "C" into hearing aid.

**BiCROS mode:**  
Slide plug "B" into hearing aid.


### UM SERIES

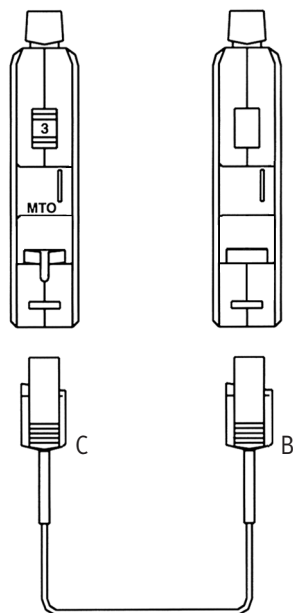

**CROS mode:**  
Slide plug "C" into hearing aid.

**BiCROS mode:**  
Slide plug "B" into hearing aid.


### US SERIES

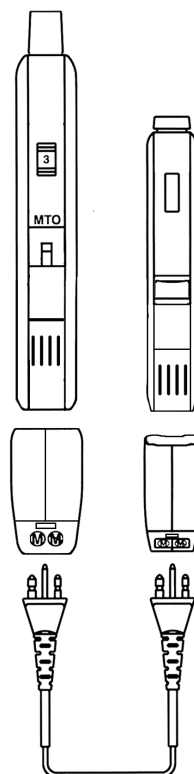

**CROS mode:**  
Connect US series CROS shoe (identified by a beige rectangle) to hearing aid. Connect ICON CROS shoe to microphone unit. Connect cord to shoes.

**BiCROS mode:**  
Connect US series BiCROS shoe (identified by a brown rectangle) to hearing aid. Connect ICON BiCROS shoe to microphone unit. Connect cord to shoes.


### ICON & SOUND F/X SERIES

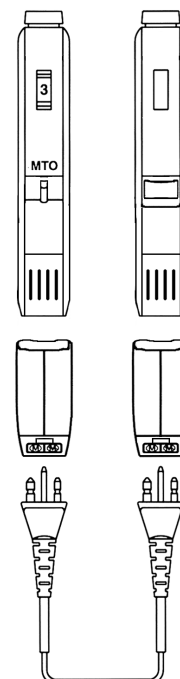

**CROS mode:**  
Both ICON and Sound F/X hearing aids use ICON CROS shoes (identified by a beige rectangle). Connect cord to shoes.

**BiCROS mode:**  
Both ICON and Sound F/X hearing aids use ICON BiCROS shoes (identified by a brown rectangle). Connect cord to shoes.

### UE AND UM SERIES:

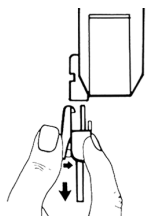

To connect or remove the Cros/BiCros plug, squeeze the latch lever and slide the plug into or out of the slot in the receptacle. Align the metal connector with the correct slot.

The offside microphone unit is shown on the right in each case.

### US, ICON AND SOUND F/X SERIES:

The offside microphone unit, depicted on the right in each diagram, is housed in ICON or Sound F/X casing.

# CROS/BiCROS – Conventional Products

## HEARING AIDS

- To use CROS and BiCROS, hearing aids must have audio input capability. We offer the following conventional product series for CROS and BiCROS use: UE, UM, US, ICON and Sound F/X.
- UE and UM series use CROS/BiCROS cords.
- US, ICON and Sound F/X hearing aids use CROS or BiCROS shoes with a cord.
- All models operate normally when CROS/BiCROS cords and/or shoes are removed.

## MICROPHONE UNIT

**UE Series:** Available in beige or brown (for use with UE hearing aids only).

**UM Series:** Available in beige, brown or grey (for use with UM hearing aids only).

**ICON and Sound F/X Series:** Available in beige, brown, taupe or grey (for use with US, ICON and Sound F/X hearing aids).

## UE AND UM SERIES CORDS


One connecting cord serves for both CROS and BiCROS modes.

**Mode of Use:** **CROS** - Slide plug "C" into hearing aid.

**BiCROS** - Slide plug "B" into hearing aid.

**Cord Lengths:** 14.5 cm/5.75" 17.5 cm/6.88" 20.5 cm/8.0" 23.5 cm/9.25" 26.5 cm/10.44"  
16.0 cm/6.28" 19.0 cm/7.5" 22.0 cm/8.68" 25.0 cm/9.88" 28.0 cm/11.0"

Cord length measurement should be made from ear to ear, behind the head.

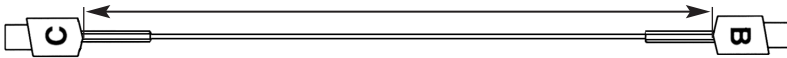

## US, ICON AND SOUND F/X SERIES CORDS AND SHOES

Two CROS shoes or BiCROS shoes and a cord are required.

**Mode of Use:** **CROS** - Connect CROS shoes to hearing aid and microphone unit. Connect cord to shoes.


**BiCROS** - Connect BiCROS shoes to hearing aid and microphone unit. Connect cord to shoes.

**CROS Shoe:** Identified by a beige rectangle.

**BiCROS Shoe:** Identified by a brown rectangle.

**Cord Lengths:** 14.5 cm/5.75" 17.5 cm/6.88" 20.5 cm/8.0" 23.5 cm/9.25" 26.5 cm/10.44"  
16.0 cm/6.28" 19.0 cm/7.5" 22.0 cm/8.68" 25.0 cm/9.88" 28.0 cm/11.0"

Cord length measurement should be made from ear to ear, behind the head.

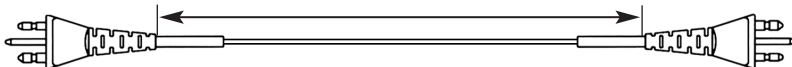

### CORPORATE OFFICE/INTERNATIONAL

Kitchener, Ontario, Canada  
877 492 6244; 519 895 0100  
fax 519 895 0108

### U.S.A.

Plymouth, Minnesota  
800 888 8882; 763 744 3300  
fax 763 557 8828

### CANADA

Cambridge, Ontario  
519 650 9111; 800 265 8255  
fax 800 949 6663

### EUROPE

Bremen, Germany  
49 421 43 87 90  
fax 49 421 48 81 56

