

Direct Audio Input



INDIGO/ELEMENT/
LIAISON/CONVERSA.NT/
UNISON 6/3/E



UNISON 6/3/E HP



US 80
SERIES

INDIGO™ / ELEMENT™ / LIAISON™ / CONVERSA.NT™ / UNISON™ 6/3/E AUDIO INPUT UNIT FEATURES

- Replace battery door with optional direct audio input battery door unit (see instructions on reverse)
- Integrated contacts for superior reliability and reduced interference
- Can be left attached to the hearing aid indefinitely
- Audio input is programmed with the Direct Audio Input environment in the fitting software
- Compatible with Phonak MLx system
- Suitable for use with Phonic Ear, Phonak, Telex and other FM systems, lapel microphones and many other audio applications
- Receptacle: 3 pin polarized plug (Europlug)

PROGRAMMING INDIGO™ / ELEMENT™ / LIAISON™ / CONVERSA.NT™ / UNISON™ 6/3/E FOR DIRECT AUDIO INPUT USE

- To program a hearing aid for use with audio input, select the Direct Audio Input (DAI) environment for the desired program(s) in the fitting software
- To achieve audio signal only (hearing aid microphone off), choose DAI for the input option in the DAI program
- For audio plus hearing aid (hearing aid microphone on), select Mic+DAI as the input option

US 80 – SHOE FEATURES

- Compatible with Unitron Hearing direct audio input hearing aids
- Conveniently located two-position microphone switch



M = hearing aid microphone on (audio input signal and hearing aid microphone function simultaneously)

Ⓜ = hearing aid microphone off (audio input signal only)

- Compatible with Phonak MLx system
- Suitable for use with Phonic Ear, Phonak, Telex and other FM systems, lapel microphones and many other audio applications
- Receptacle: 3 pin polarized plug (Europlug)

Direct Audio Input

AUDIO SHOES

Indigo/Element/Liaison/Conversa.NT/Unison 6/3/E

054-5123-01 (beige)

054-5123-13 (clear)

Unison 6/3/E High Power

054-5119-01 (beige)

054-5119-13 (clear)

US8o

054-5027-01 (beige)

BATTERY DOOR REMOVAL TOOLS

054-5116 battery size 13

054-5120 battery size 675

AUDIO CORDS

Auditory Training Systems

(may not be compatible with all FM systems)

For Phonic Ear Model 475 & Telex FM Systems

057-0053 Single (60 cm/23.5", 2-pin)

057-0050 Y-Cord (60 cm/23.5", 2-pin)

For Phonic Ear Solaris or Phonak Micro Vox

052-5002 Single (60 cm/23.5", 3-pin)

052-5003 Single (75 cm/29.5", 3-pin)

052-5004 Y-Cord (60 cm/23.5", 3-pin)

052-5005 Y-Cord (75 cm/29.5", 3-pin)

General Application

3.5 mm plug, mono, grey (TV, radio, external mic)

057-5012 Single (90 cm/35")

057-0090 Y-Cord (80 cm/31.5")

3.5 mm plug, stereo, attenuated, beige (Walkman, Phonic Ear Easy Listener)

054-5037 Single (52 cm/20.5")

054-5038 Y-Cord (52 cm/20.5")

TO EQUIP INDIGO™ / ELEMENT™ / LIAISON™ / CONVERSA.NT™ / UNISON™ 6/3/E BTES FOR DIRECT AUDIO INPUT APPLICATIONS:

Replace the battery door with optional direct audio input battery door unit.

