



Moda™
Digital BTE
Guide

Table of Contents

Your Moda™ Digital Hearing Aids	1
Practical Solutions to Everyday Problems	2
Getting the Most Out of Your Moda™ Hearing Aids	3
Using the Moda™ Digital BTE Guide.....	4
Moda™ BTEs with Slim Tubes.....	5
Moda™ BTEs with Earhooks	5
Turning Your Moda™ Hearing Aids On and Off.....	5
Replacing the Battery	6
Low Battery Warning.....	7
Caring for Batteries.....	7
Operating Instructions	8
Moda™ BTEs with Earhook - Earmolds	10
Moda™ BTEs with Slim Tubes - Cleaning the Slim Tubes & Domes.....	11
Using Moda™ with the Telephone	12
Assistive Listening Devices	13
Caring for Your Moda™ Hearing Aids.....	14
Warnings	15
Troubleshooting Guide.....	18
Warning to Hearing Aid Dispensers	20

Your Moda™ Digital Hearing Aids

Hearing Healthcare Professional: _____

Telephone: _____

Model: _____

Serial Number: _____

Replacement Batteries: Size 10

Warranty: _____

Use Program 1 for: _____

Use Program 2 for: _____

Use Program 3 for: _____

Date of Purchase: _____

Practical Solutions to Everyday Problems

Congratulations on choosing Moda™ BTE digital hearing instruments, the new ultra-small, stylish and fully-featured BTE. For 40 years, Unitron Hearing has been committed to making life better for people with hearing loss. This means a commitment to developing high-quality hearing solutions that incorporate special features to solve the everyday problems and concerns you have with hearing loss and hearing aids.

Moda BTE digital hearing instruments offer a cosmetically appealing solution to satisfy a wide range of hearing losses. Moda's exceptionally tiny profile is inconspicuous and includes advanced features so you can enjoy more natural sound and understand conversations in a wide variety of listening situations.

Getting the Most Out of Your Moda™ Hearing Aids

Adjusting to your new hearing instruments will take some time. In the beginning it is important that you do not use the hearing aids for longer than is comfortable. Depending on your previous experience with hearing instruments, a few hours a day may be enough and then you can increase wearing time gradually. Once you have become accustomed to your hearing aids, you should wear them all day every day since infrequent use will not permit you to adapt to your hearing aids and enjoy their full benefits. The quicker you get used to the everyday sounds around you, the less you will notice that you are wearing hearing instruments.

Using the Moda™ Digital BTE Guide

Refer to the table of contents for a complete listing of the topics covered in this guide.

Your Moda BTE hearing instruments come equipped with either slim tubes or with standard earhooks. Refer to the diagrams below to identify which type of BTE you have and some of the components on your Moda hearing instruments.

Moda with Slim Tube



Legend

- 1 Microphone and Microphone Shield
- 2 Program Button
- 3 Slim Tube
- 4 Battery Door/On & Off Switch
- 5 Dome
- 6 Retention Piece

Moda with Earhook



Legend

- 1 Earhook
- 2 Microphone and Microphone Shield
- 3 Program Button
- 4 Battery Door/On & Off Switch

Moda™ BTEs with Slim Tubes

Putting Moda™ on Your Ear

1. Place the Moda hearing instrument over the top of your ear.
2. Hold the slim tube where it attaches to the dome and gently push the dome into your ear canal. The slim tube should lie flush against your head and not stick out.
3. Place the retention piece in your ear so it rests at the bottom of the opening of your ear canal.

Moda™ BTEs with Earhooks

Putting Moda™ on Your Ear

1. Place the Moda hearing instrument over the top of your ear.
2. Carefully insert the earmold in your ear. The earmold should fit into your ear snugly and comfortably.

Turning Your Moda™ Hearing Aids On and Off

Your Moda hearing instruments have a three-position battery door that acts as an off/on switch and that allows access to the battery compartment.

To turn the hearing aid on, close the battery door fully.

To turn the hearing aid off, partially open the battery

door. This position will also allow excess moisture to vent away from the battery compartment.

To replace the hearing aid battery, fully open the battery door for access to the battery compartment.

Replacing the Battery

1. Gently swing out the battery compartment door with your fingernail.
2. Grasp the battery with your thumb and index finger and remove.
3. Insert the new 10 battery into the battery compartment with the plus (+) sign on the battery facing the same way as the plus (+) sign on the edge of the battery door. This will ensure that the battery door closes properly.

Note: If the battery is inserted incorrectly, the door will not close.

4. Close the battery compartment door.

Low Battery Warning

When you hear one long beep, your digital hearing aids are warning you that their batteries are low. You will hear the warning approximately every 30 minutes until you change the batteries or the batteries die. After the first warning, you may experience some reduction in sound quality. This is normal and can be remedied by inserting fresh batteries in the hearing aids. If you prefer, your hearing healthcare professional can change the pitch and loudness of the low battery beep or turn it off entirely.

Caring for Batteries

- Always discard batteries carefully.
- To prolong battery life, remember to turn your hearing aids off when not in use.
- Remove the batteries and keep the battery door open while hearing aids are not in use. This will allow internal moisture to evaporate.

Warnings

- Never leave hearing aids or batteries where small children and pets can reach them.
- Never put hearing aids or batteries in your mouth. If a hearing aid or battery is swallowed, call a physician immediately.

Operating Instructions

Changing Programs on Your Moda™ Digital Hearing Aids

Your Moda digital hearing aids may have up to three programs. Each program has been individually tailored by your hearing healthcare professional to meet your particular listening needs.

Moda has a push-button for switching between programs. Each time you push the program button, you will move to a new program. Your hearing aids beep to indicate which program you are in (i.e. one beep for program 1, two beeps for program 2, etc.) If you prefer, your hearing healthcare professional can also adjust your hearing aids so you do not hear the beep.

Volume Control

The volume on your Moda hearing aids adjusts automatically in reaction to the loud or quiet sounds around you.

Wind Noise Manager

Wind travelling over the microphone inlets on your hearing aids causes Moda's wind noise manager to engage automatically based on whether wind conditions are moderate or high. When wind noise manager is engaged, sounds such as speech, may become quieter because the wind noise manager is

working to reduce the loud noise produced by the wind energy. When you are no longer in a windy environment, the wind noise manager will not be active and desirable sounds, such as speech, will once again become louder.

Start Up Mute

Moda digital hearing instruments have an optional start up mute feature that can be activated by your hearing healthcare professional. If the start up mute is activated, your hearing aids will not turn on until the program button is pushed, even if the battery door is in the ON position. The hearing aids return to the start up mute position each time you turn your hearing aids on. This allows you to insert your hearing aids without experiencing whistling.

Adaptive Directional Microphones

Moda digital hearing instruments may contain adaptive directional microphones that are activated by the push of the program button. These directional microphones focus on sounds in front of you (e.g., speech) while reducing sounds from the sides or behind you (e.g., noise). Even if the sounds are continually moving, the directional microphone system intuitively adapts to reduce the background noise. Your hearing healthcare professional will tell you which program(s) contain the adaptive directional microphones.

Microphone Protectors

Your Moda hearing aids have a shield to protect the microphones from dirt and debris. Debris on the microphone protector reduces the sound quality of the instrument. You should have the microphone protectors replaced by your hearing healthcare professional approximately every three months to ensure optimal sound quality.

Moda™ BTEs with Earhook - Earmolds

If your Moda hearing instruments have an earhook, (see Moda™ BTEs with Earhook), your hearing healthcare professional will fit you with customized earmolds. Earmolds feed amplified sound from the hearing aids into the ears. They must fit into your ears snugly and comfortably. If amplified sound leaks out of your ear, you may hear whistling.

Always ensure earmolds are clean and free of ear wax and moisture. Do not use alcohol to clean your earmolds. If the earmolds become plugged, clear the opening with a wax loop or pipe cleaner. If your physician prescribes eardrops, clean any moisture that may get into the earmolds or tubing to prevent plugging. If the earmolds require further cleaning, disconnect the plastic tube from the hook of the hearing aids. Wash only the earmolds in warm water with a mild soap.

Rinse them with cool water and allow them to dry overnight. Make sure the earmold tubes are dry before reconnecting them to each hook on your hearing aids.

Moda™ BTEs with Slim Tubes - Cleaning the Slim Tubes & Domes

You should have the slim tubes and domes replaced by your hearing healthcare professional approximately every three months or when they become stiff or brittle. Clean the domes daily with a damp cloth. You should also clean the slim tubes periodically, with the cleaning pin provided, when you begin to notice debris in and around the tubes.

To clean the slim tubes and domes:

1. Hold the slim tube in one hand and the Moda hearing instrument in the other hand.
2. Gently turn the Moda hearing instrument until it detaches from the slim tube.
3. Use a damp cloth to clean the outside of the slim tube and the dome.
4. Using the black cleaning pin provided in the kit, gently insert the cleaning pin where the slim tube attaches to the hearing instrument and push the pin all the way through the tube and out through the dome.

Note: The slim tubes and domes should never be rinsed or submerged in water as water drops may

become lodged in the tube, block sound or damage the electrical components of the hearing instrument.

5. Once the slim tube has been cleaned, reattach the slim tube to the Moda hearing instrument.

Using Moda™ with the Telephone

Moda has the ability, depending on hearing loss, vent size and style of hearing aid to provide feedback-free phone use without program changes. For many wearers this means, when the phone rings, all you have to do is pick up the telephone and hold it to your ear normally. In some situations when using a cell phone, you may experience digital interference that sounds like static, buzzing or beeping. If you experience interference, increase the distance between your hearing instrument and the phone receiver.

Moda also comes equipped with a telecoil that can help you listen on the telephone. The telecoil program is selected with the program button. When using the telecoil, center the telephone receiver on the hearing aid and move the receiver along the hearing aid to locate the clearest, strongest signal. To return to normal listening, push the program button. Your hearing aid will beep to indicate which program you are in.

Assistive Listening Devices

Loop Systems

Moda's telecoil can also help you listen in public places equipped with telecoil compatible assistive listening devices such as a loop system. If your hearing aids do not operate when the telecoil is on, the loop system may not be operating or you may be in a "dead" spot. Try positioning yourself in a different part of the looped area or return your hearing aids to the microphone position and sit as close to the speaker as possible.

FM and Infrared Systems

Some FM systems consist of a teleloop worn around the neck, which is connected to the FM receiver worn on a belt or in a shirt pocket. To listen through the FM system, select the telecoil program and turn on the FM receiver. To keep the signal clear, you may need to adjust the volume control on the FM receiver.

With some infrared systems you may need to remove your hearing aids and wear the system's headset receiver. If the volume on the infrared receiver is not loud enough, use your hearing aids instead.

Caring for Your Moda™ Hearing Aids

- Never use alcohol to clean your hearing aids, earmolds, the slim tubes or domes.
- Do not use sharp tools to dislodge ear wax. Sticking household items into your hearing aids, earmolds, the slim tubes or domes can seriously damage them. See “Earmolds and Cleaning the Slim Tubes & Domes”.
- Always remove your hearing aids when using hair care products. The hearing aids can become clogged and cease to function properly.
- Protect your hearing aids from excessive heat (hair dryer, vehicle glove box or dashboard) and moisture (bath or shower). Do not immerse the hearing aids in water.
- If your hearing aids do become wet, do not attempt to dry them in an oven or microwave. Do not adjust any controls. Open the battery doors immediately, and allow your hearing aids to dry naturally for 24 hours.
- Regular use of a dehumidifier, such as a Dri-Aid kit, can help prevent corrosion. See your hearing healthcare professional for more information.
- Do not drop your hearing aids or knock them against hard surfaces.

Warnings

- Hearing aids should only be used as directed by your physician or hearing healthcare professional.
- Hearing aids will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions.
- Do not use your hearing aids in explosion hazard areas.
- Allergic reactions to hearing aids are unlikely. However, if you experience itching, redness, soreness, inflammation or a burning sensation in or around your ears, inform your hearing healthcare professional and contact your physician.
- Remove your hearing aids for CAT and MRI scans or for other electromagnetic procedures.

Precautions

- The use of hearing aids is only part of hearing rehabilitation; auditory training and lip reading instruction may be required as well.
- In most cases, infrequent use of hearing aids does not provide full benefit. Once you have become accustomed to your hearing aids, wear your hearing aids everyday all day.
- Your hearing aids use the most modern components to provide the best possible sound quality in every

listening situation. However, communication devices such as digital cell phones can create interference (a buzzing sound) in hearing aids. If you experience interference from a cell phone being used close by, you can minimize this interference in a number of ways. Switch your hearing aids to another program, turn your head in a different direction or locate the cell phone and move away from it.

Labelling

The serial number and year of manufacture are located inside the battery door.

Troubleshooting Guide

SYMPTOM	CAUSE	POSSIBLE REMEDY
No sound	<ul style="list-style-type: none">• Not turned on• Low/dead battery• Poor battery contact • Battery upside down• Earmolds/slim tubes blocked with ear wax • Plugged microphone protector	<ul style="list-style-type: none">• Turn on• Replace battery• Consult your hearing healthcare professional• Insert battery plus (+) side up• Clean earmolds. See “Earmolds”. Use cleaning pin to dislodge earwax in slim tubes. See “Cleaning the Slim Tubes & Domes”. Consult your hearing healthcare professional.• Consult your hearing healthcare professional
Not loud enough	<ul style="list-style-type: none">• Low volume • Low battery• Earmolds/slim tubes not inserted properly• Change in hearing • Earmolds/slim tubes blocked with ear wax • Plugged microphone protector	<ul style="list-style-type: none">• See hearing healthcare professional• Replace battery• See “Earmolds” or “Putting Moda on Your Ear”. Reinsert carefully• Consult your hearing healthcare professional• Clean earmolds. See “Earmolds”. Use cleaning pin to dislodge earwax in slim tubes. See “Cleaning the Slim Tubes & Domes”. Consult your hearing healthcare professional.• Consult your hearing healthcare professional
Intermittent	<ul style="list-style-type: none">• Low battery• Dirty battery contact	<ul style="list-style-type: none">• Replace battery• Consult your hearing healthcare professional

SYMPTOM	CAUSE	POSSIBLE REMEDY
Whistling	<ul style="list-style-type: none"> • Earmolds/slim tubes not inserted properly • Hand/clothing near ear • Poorly fitting earmolds/slim tubes 	<ul style="list-style-type: none"> • Remove and reinsert • Remove hand/clothing from ear • Consult your hearing healthcare professional
Not clear, distorted	<ul style="list-style-type: none"> • Poorly fitting earmolds/slim tubes • Earmolds/slim tubes blocked with ear wax • Low battery • Plugged microphone protector 	<ul style="list-style-type: none"> • Consult your hearing healthcare professional • Clean earmolds. See “Earmolds”. Use cleaning pin to dislodge earwax in slim tubes. See “Cleaning the Slim Tubes & Domes”. Consult your hearing healthcare professional. • Replace battery • Consult your hearing healthcare professional
Earmolds/slim tubes falling out of ear	<ul style="list-style-type: none"> • Poorly fitting earmolds/slim tubes • Earmolds/slim tubes not inserted properly 	<ul style="list-style-type: none"> • Consult your hearing healthcare professional • See “Earmolds” or “Putting Moda™ on Your Ear”. Reinsert carefully.
"Motorboating" sound	<ul style="list-style-type: none"> • Low battery • Dirty battery contact 	<ul style="list-style-type: none"> • Replace battery • Consult your hearing healthcare professional
Weak on the telephone	<ul style="list-style-type: none"> • Telephone not positioned properly • Hearing aid requires adjustment 	<ul style="list-style-type: none"> • Move telephone receiver around ear for clearer signal • Consult your hearing healthcare professional
One long beep	<ul style="list-style-type: none"> • Low battery 	<ul style="list-style-type: none"> • Replace battery

For problems not listed in the guide, contact your hearing healthcare professional. If you do not have a hearing healthcare professional, please contact the nearest office listed on the back page of this booklet.

Warning to Hearing Aid Dispensers

A hearing aid dispenser should advise a prospective hearing aid user to consult promptly with a licensed physician (preferably an ear specialist) before dispensing a hearing aid if the hearing aid dispenser determines through inquiry, actual observation, or review of any other available information concerning the prospective user, that the prospective user has any of the following conditions: (i) Visible congenital or traumatic deformity of the ear. (ii) History of active drainage from the ear within the previous 90 days. (iii) History of sudden or rapidly progressive hearing loss within the previous 90 days. (iv) Acute or chronic dizziness. (v) Unilateral hearing loss of sudden or recent onset within the previous 90 days. (vi) Audiometric air-bone gap equal to or greater than 15 decibels at 500 hertz (Hz), 1,000 Hz, and 2,000 Hz. (vii) Visible evidence of significant cerumen accumulation or a foreign body in the ear canal. (viii) Pain or discomfort in the ear. Special care should be exercised in selecting and fitting a hearing aid whose maximum sound pressure level exceeds 132 decibels because there may be risk of impairing the remaining hearing of the hearing aid user. (This provision is required only for those hearing aids with a maximum sound pressure capability greater than 132 decibels (dB).)

Important Notice for Prospective Hearing Aid Users

Good health practice requires that a person with a hearing loss have a medical evaluation by a licensed physician (preferably a physician who specializes in diseases of the ear) before purchasing a hearing aid.

Licensed physicians who specialize in diseases of the ear are often referred to as otolaryngologists, otologists or otorhinolaryngologists. The purpose of medical evaluation is to assure that all medically treatable conditions that may affect hearing are identified and treated before the hearing aid is purchased. Following the medical evaluation, the physician will give you a written statement that states that your hearing loss has been medically evaluated and that you may be considered a candidate for a hearing aid. The physician will refer you to an audiologist or a hearing aid dispenser, as appropriate, for a hearing aid evaluation. The audiologist or hearing aid dispenser will conduct a hearing aid evaluation to assess your ability to hear with and without a hearing aid. The hearing aid evaluation will enable the audiologist or dispenser to select and fit a hearing aid to your individual needs. If you have reservations about your ability to adapt to amplification, you should inquire about the availability of a trial-rental or purchase-option program. Many

hearing aid dispensers now offer programs that permit you to wear a hearing aid for a period of time for a nominal fee after which you may decide if you want to purchase the hearing aid. Federal law restricts the sale of hearing aids to those individuals who have obtained a medical evaluation from a licensed physician. Federal law permits a fully informed adult to sign a waiver statement declining the medical evaluation for religious or personal beliefs that preclude consultation with a physician. The exercise of such a waiver is not in your best health interest and its use is strongly discouraged.

Children With Hearing Loss

In addition to seeing a physician for a medical evaluation, a child with a hearing loss should be directed to an audiologist for evaluation and rehabilitation since hearing loss may cause problems in language development and the educational and social growth of a child. An audiologist is qualified by training and experience to assist in the evaluation and rehabilitation of a child with a hearing loss.

UNITRON HEARING DISTRIBUTORS

CORPORATE OFFICE/INTERNATIONAL

20 Beasley Drive, P.O. Box 9017, Kitchener, ON N2G 4X1 Canada

CANADA

5 Cherry Blossom Road, Unit 2, Cambridge, ON N3H 4R7

EUROPE

Daimlerstrasse 22, 70736 Fellbach, Germany

FRANCE

2, rue Maryse Bastié, 69500 Bron, France

NETHERLANDS

Archimedesbaan 19, P.O. Box 1214, 3430 BE Nieuwegein

UNITED KINGDOM

St. George House, Cygnet Court, Centre Park, Warrington, Cheshire WA1 1PD

U.S.A.

2300 Berkshire Lane North, Plymouth, MN 55441

MANUFACTURER

UNITRON HEARING LTD.

20 Beasley Drive, P.O. Box 9017, Kitchener, ON N2G 4X1 Canada

CE
0543

unitronTM
hearing 



DISTRIBUTOR